

Seat Assembly Removal / Installation

NOTE: These instructions are usually the first step if you have to work on or replace the seat assembly.

Tools Required

Phillips screwdriver

3/8" wrench

CAUTION

The TRS 4000 seat assembly is heavy; it weighs 50 lbs (23 kg). To avoid injury or damage to the product, use caution when removing or installing. Obtain assistance as required. Use proper lifting technique.

1. Remove seat rail cap.



2. Remove seat rail bumpers.



3. Roll seat off of seat rail.



Keep upward pressure on front of seat to prevent scratching end of seat rail.

4. If you are going to work on the seat, turn seat upside down and lay it across rear of NuStep.



The horizontal position keeps parts from sliding out of position.

5. To install, do steps in reverse order. No photo.