

PROII00, PRO1000, and Pro 1 Wheelchair Platform Instructions

1. Unplug the power cord, if applicable.
2. Slide the seat assembly off the rear of the unit.
3. Lift the PRO unit up until it rests on its nose.
4. If the unit is a PRO1000 or a PRO II, loosen the four (4) $\frac{1}{2}$ " bolts that are located inside the frame and closest to the seat end on each side. These should be loosened approximately four (4) turns. There are two bolts on each side and towards the front of the seat track that will not be loosened. This will leave enough room to slide the short leg of the platform, with the slots between the lock washer and stainless steel seat track. **Caution must be taken not to loosen bolts enough to disengage from the seat rails.**
5. If the unit is a PRO 1, loosen the four (4) $\frac{1}{2}$ " bolts, located inside the frame in the center on each side, approximately four (4) turns. On each side, there are two bolts that are not loosened, one towards the front and one at the rear of the seat track. This will leave enough room to slide the short leg of the platform, with the slots between the lock washer and stainless steel seat track. **Caution must be taken not to loosen bolts enough to disengage from the seat rails.**
6. When platform slots are between the lock washer and stainless steel seat track, push the platform up until it is flush with the bottom of the black frame of the unit. Tighten the bolts.
7. Please call Scifit Service @ 800-745-1373 if there are any questions or problems.

